

# 1500/1712/2200G

## N95 HEALTHCARE PARTICULATE RESPIRATOR AND SURGICAL MASK

- Highest ASTM level of fluid resistance on the market (160mm Hg)
- Bacterial Filtration Efficiency (BFE)>99.9%
- Meets CDC guidelines for TB exposure control standards and protection against laser plumes of surgical smoke
- No need for pre-stretching straps
- Not manufactured with natural rubber latex

### 1500 N95 Series – Available in 5 sizes for today's diverse workforce



1510 N95 XS




1511 N95 S

NSN #4240-01-496-0850



1512 N95 M

NSN #4240-01-496-0916

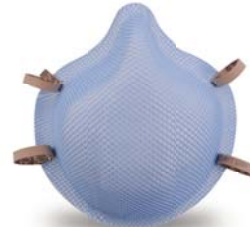

1513 N95 L

NSN #4240-01-496-0918



1517 N95 LP

### 2200G N95 Series – Suggested for stockpiling



2210GN95-XS



2211GN95-S



2212GN95-M/L



2217GN95-LP

### 1712N95 – FastFit 2-Strap Flat Fold

- Flat fold convenience with cup shaped comfort
- Quick and easy to put on, unlike all other flat folds
- Individually packaged for hygiene in a dispenser style box



1712N95



Flat fold fits in pocket



**MOLDEX**  
Ideas that wear well.®

# 1500/2200G/1712

## N95 HEALTHCARE PARTICULATE RESPIRATOR AND SURGICAL MASK

### WARNING TO USER

- Non-latex straps are used in the headbands of these respirators. However, individuals highly sensitive to natural rubber latex may have an allergic reaction.
- This product does not eliminate the risk of contracting any disease or infection.** Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.
- This product has not been sold with warnings or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact the Moldex Technical Services Department at +1 (310) 837-6500 or +1 (800) 421-0668, ext. 512/550.

### DESCRIPTION

- The Moldex Healthcare N95 Particulate Respirator and Surgical Mask is designed to help provide respiratory protection for the wearer. This product has been tested<sup>1</sup> and certified by NIOSH as an N95 respirator and as having a filter efficiency level of 95% or greater against particulate aerosols free of oil. It is fluid resistant,<sup>2</sup> disposable and may be worn in surgery or throughout the hospital.
- This product contains no components made from natural rubber latex.

<sup>1</sup> Tested in accordance with NIOSH 42 CFR 84

<sup>2</sup> Passed ASTM F 1862 @ 160mm Hg

### INTENDED USE

The various models of Moldex Type N95 Healthcare Particulate Respirators and Surgical Masks meet CDC Guidelines for TB Exposure Control within healthcare facilities. These devices are also intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids and particulate material.

### CONTRAINDICATIONS

- Not for use with beards, or other facial hair that prevents direct contact between the face and sealing surface of the respirator.
- Eyewear must not prevent direct contact between the face and sealing surface of the respirator.
- Not to be used on children.


### RESTRICTIONS

- Before use, a written respiratory protection program in accordance with 29 CFR 1910.134 must be implemented.
- This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
- Prior to each use, carefully inspect the entire respirator, including filter media and strap attachment area for tears and damage. Staple or strap attachment perforations do not effect NIOSH approval.
- If respirator is damaged, distorted, a proper fit cannot be obtained or breathing becomes difficult, leave contaminated area as soon as possible and replace respirator.
- If the respirator comes in contact with blood or fluids, including body fluids, leave contaminated area as soon as possible and discard and replace the respirator.
- Do not alter, modify or abuse this respirator.
- Store unused respirators in box/bag in a clean, dry, non-contaminated area.
- Dispose of respirator according to your employer's policy and local regulations.
- Do not reuse or store for reuse or hang around neck unless your employer specifically authorizes reuse.
- When used for surgical procedures, discard after every use.
- Use respirator before the "use by" expiration date printed on box/bag.




**MOLDEX-METRIC, INC.**  
 10111 Jefferson Blvd.  
 Culver City, CA 90232  
 TEL: +1 (800) 421-0668 or  
 +1 (310) 837-6500  
 FAX: +1 (310) 837-9563  
 E-mail: sales@moldex.com  
 www.moldex.com

**CANADIAN CUSTOMER SERVICE**  
 Tel: +1 (800) 421-0668, Ext. 550/512  
 Fax: +1 (310) 837-9563



Moldex-Metric, Inc.  
 CULVER CITY, CALIFORNIA, U.S.A.  
 800/421-0668



THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

| TC-      | PROTECTION <sup>1</sup> | RESPIRATOR |      |      |      |      | CAUTIONS AND LIMITATIONS <sup>2</sup> |
|----------|-------------------------|------------|------|------|------|------|---------------------------------------|
|          |                         | 1510       | 1511 | 1512 | 1513 | 1517 |                                       |
| 84A-0013 | N95                     |            | X    | X    | X    | X    | A B C J M N O P                       |
| 84A-5171 | N95                     | X          |      |      |      |      | A B C J M N O P                       |

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

| TC-      | PROTECTION <sup>1</sup> | RESPIRATOR  |            |              |             | CAUTIONS AND LIMITATIONS <sup>2</sup> |
|----------|-------------------------|-------------|------------|--------------|-------------|---------------------------------------|
|          |                         | 2210GN95-XS | 2211GN95-S | 2212GN95-M/L | 2217GN95-LP |                                       |
| 84A-4339 | N95                     | X           | X          | X            | X           | A B C J M N O P                       |

THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

| TC-      | PROTECTION <sup>1</sup> | RESPIRATOR | CAUTIONS AND LIMITATIONS <sup>2</sup> |
|----------|-------------------------|------------|---------------------------------------|
|          |                         | 1712N95    |                                       |
| 84A-5227 | N95                     | X          | A B C J M N O P                       |

1. Protection

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.  
 B- Not for use in atmospheres immediately dangerous to life or health.  
 C- Do not exceed maximum use concentrations established by regulatory standards.  
 J- Failure to properly use and maintain this product could result in injury or death.  
 M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.  
 N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.  
 O- Refer to User's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.  
 P- NIOSH does not evaluate respirators for use as surgical masks.

| Product Code | Size | Valve Type | Strap Material | Display Quantity | Case Quantity |
|--------------|------|------------|----------------|------------------|---------------|
| 1510 N95     | XS   | None       | Non-Latex      | 20/Box           | 8 Boxes       |
| 1511 N95     | S    | None       | Non-Latex      | 20/Box           | 8 Boxes       |
| 1512 N95     | M    | None       | Non-Latex      | 20/Box           | 8 Boxes       |
| 1513 N95     | L    | None       | Non-Latex      | 20/Box           | 8 Boxes       |
| 1517 N95     | LP   | None       | Non-Latex      | 20/Box           | 8 Boxes       |

| Product Code | Size | Valve Type | Strap Material | Display Quantity | Case Quantity |
|--------------|------|------------|----------------|------------------|---------------|
| 2210GN95-XS  | XS   | None       | Non-Latex      | 20/Box           | 12 Boxes      |
| 2211GN95-S   | S    | None       | Non-Latex      | 20/Box           | 12 Boxes      |
| 2212GN95-M/L | M/L  | None       | Non-Latex      | 20/Box           | 12 Boxes      |
| 2217GN95-LP  | LP   | None       | Non-Latex      | 20/Box           | 12 Boxes      |

| Product Code | Size | Valve Type | Strap Material | Display Quantity | Case Quantity |
|--------------|------|------------|----------------|------------------|---------------|
| 1712N95      | M/L  | None       | Non-Latex      | 20/Box           | 10 Boxes      |

### LIMITED WARRANTY IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. Exclusive Remedies: damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex, products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

### MOLDEX TECHNICAL ASSISTANCE

For more information call +1 (800) 421-0668 or +1 (310) 837-6500 ext. 512/550 or E-mail: sales@moldex.com, tech@moldex.com or visit www.moldex.com.

